

Instructions for **saliva** collection

- Do not eat, drink, smoke or chew gum for 30 minutes before use.
- Before use: remove the collection funnel from the wrapper. Check that the level of the liquid in the tube is at the "2ml From" line before use. If the solution is cloudy, warm the tube gently in your hands for a few minutes.



Instructions for Collection kit, model 1 (model 2 on the back):

Step 1: Unscrew the cap from the tube being careful not to spill the liquid. Keep the cap for use later.

Screw the collection funnel on to the tube gently. **Do not over-tighten**



Step 2: Start spitting into the funnel. **Tip: If the saliva does not flow easily into the tube, unscrew the funnel partially.**

Continue spitting into the funnel until the level of liquid in the tube, not including bubbles, reaches the 4ml Fill To the line. To encourage saliva production you may rub the inside of your cheek with your tongue, or gently massage the outside of your cheeks.



Step 3: Unscrew the collection funnel and replace the tube cap tightly. Return the funnel to the wrapper and throw away.



Step 4: Invert the collection tube several times to mix the saliva with the solution.